



Turn on the ProTime[®] Monitor

- ▶ Press the **0** button.
- ▶ Watch the screen and follow the prompts.



Insert the Test Cuvette

- ▶ The barcode should be face down.
- ▶ The ProTime logo on the cuvette should face up.
- ▶ The ProTime system warms the cuvette.
- ▶ Wait for the prompt before incising the finger.




Prepare the Finger.

- ▶ Warm the hands to stimulate blood flow.
- ▶ Cleanse the finger, air dry or wipe dry.
- ▶ Wait for the prompt before incising the finger.



Incise the Finger

- ▶ Hang hand below heart level to assist capillary filling of finger.
- ▶ Place the Tenderlett[®] incision device firmly against side of finger at the 2 o'clock position on the left hand, 10 o'clock position on the right hand.
- ▶ Place thumb over the red trigger; press the red trigger.

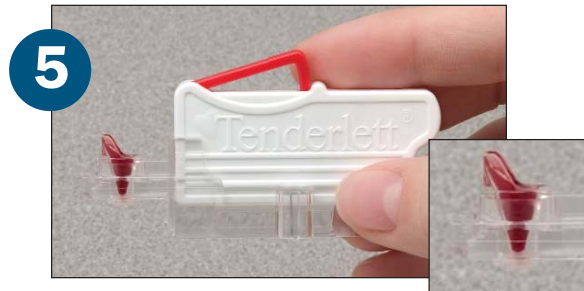


Fill the Cup to Line

- ▶ Wipe away first trace of blood.
- ▶ Gently massage the base of the finger to force blood to the tip so a large, hanging drop of blood forms.
- ▶ Touch blood drop to raised lip of cup to fill the cup all the way.



Snap the Tenderlett Plus device onto the ProTime Unit

- ▶ Hold the Tenderlett Plus at 45° angle.
- ▶ Place cup end into slot.
- ▶ Press down firmly in place, a soft click should be felt or heard.

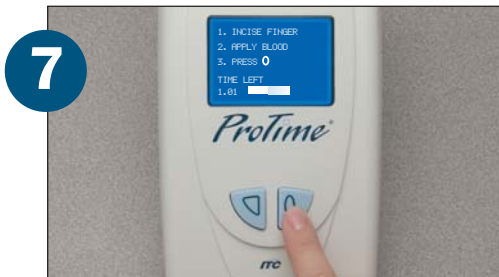


Press the 0 Button

- ▶ Press the **0** button to start the test.
- ▶ Follow the prompts on the screen.



Remove the Tenderlett Plus Device

- ▶ When PROMPTED by the window display message, remove the Tenderlett Plus device.
- ▶ The ProTime system allows you six seconds to remove Tenderlett Plus.

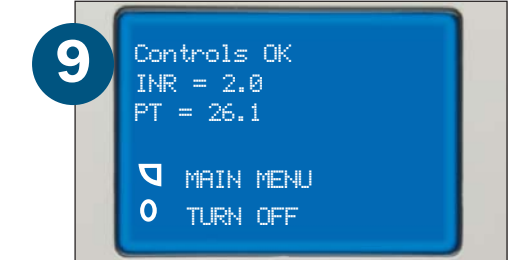

Read the Results

- ▶ The results remain displayed for five minutes.