

Test Procedure: follow the prompts on the screen.



1. TURN ON ProTime®

- Press the **0** button.
- Watch the screen and follow the prompts.



2. INSERT TEST CUVETTE

- The barcode should be face down.
- ProTime warms the cuvette.
- WAIT** for the prompt before incising the finger.




3. PREPARE THE FINGER

- Warm the hands to stimulate blood flow.
- Cleanse the finger.
- WAIT** for the prompt before incising the finger.




4. INCISE THE FINGER

- Place Tenderlett® Plus firmly against side of finger.
- Place thumb over the red trigger as shown.
- Press the red trigger.

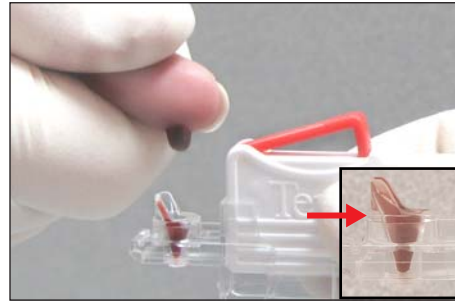


5. FILL CUP TO LINE

- Wipe away the first drop of blood.
- Form one **LARGE DROP** of blood.
- Touch drop to cup.
- FILL** to the line indicated by the arrow.

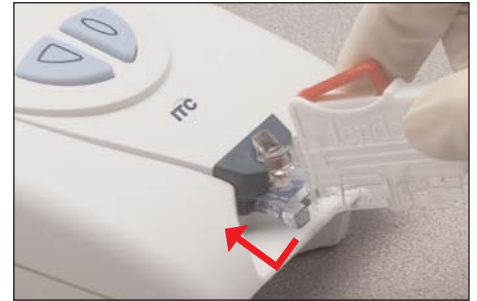

6. SNAP Tenderlett Plus ONTO ProTime

- Hold the Tenderlett Plus at an angle.
- Place cup end into slot.
- Press down to click in place.

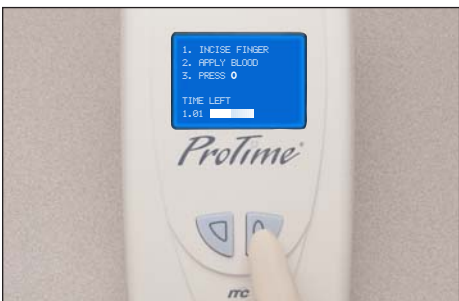


7. PRESS THE 0 BUTTON

- Press the **0** button.
- FOLLOW** the prompts on the screen.



8. REMOVE Tenderlett Plus

- When **PROMPTED** remove the Tenderlett Plus.
- ProTime allows you six seconds to remove Tenderlett Plus.

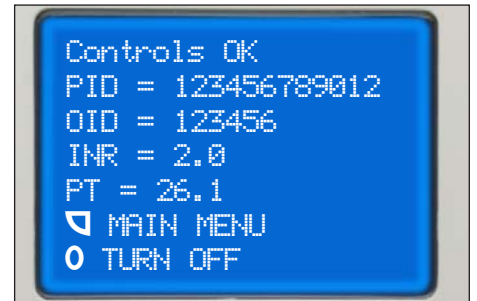

9. READ RESULTS

- The results remain displayed for five minutes.

ProTime and Tenderlett are registered trademarks of ITC.



8 Olsen Avenue • Edison, NJ 08820 USA
 tel: 732.548.5700 • fax: 732.248.1928
 www.itcmed.com
 a subsidiary of Thoratec Corporation

ITC Technical Support
 phone: 1-800-631-5945
 fax: 1-732-548-9824
 email: techsupport@itcmed.com